

Introducing... our new Twins



275 kg

Simple conversion

From 1 to 2 Lunas.
From 275 to 400 kg.
5 minutes. No tools.

Luna Twin
Bariatric solution

400 kg

Luna Twin Simple & innovative

- **400 kg bariatric solution:** The same Luna model is used for standard and bariatric lifting: 1 or 2 units.
- **Flexible design:** Quick switch from standard 275 kg to bariatric 400 kg lift in minutes. No tools.
- **Tender winner:** Selected by Central Region Denmark, largest hospital construction project in Northern Europe.
- **Cost saving investment for hospitals:** One multi purpose Luna model for all your lifting needs.

Elite Healthcare Ltd.
Phone: +353 (0) 1 8903932
Fax: +353 (0) 1 8903937

Web: www.elitehealthcare.ie
Email: info@elitehealthcare.ie

Dublin: F1 Swords Enterprise Park,
Feltrim Road, Swords, Co Dublin,
K67 E984, Ireland.

Cork: 602 Harbour Point Business
Park, Little Island, Cork, T45 X624,
Ireland.

New Luna Twin

5 minute installation. No tools.




- ①
- Mount the second Luna Twin on the track.
 - Lower one of the straps on the hand control, so both straps are equal in length.
 - Attach the distance rod between the two Luna units.
 - Connect the twin cable to the Luna units.
 - Reset the system by pressing "STOP" and then press and hold "R" for 5 sec.

- ②
- Attach the distance bar on the straps. If needed, adjust the strap height with the hand control by pressing right or left.

- ③
- Attach the spreader bar with the quick lock system.

- &
- Weigh scale and additional side spreader bars for large patients are available as accessories.

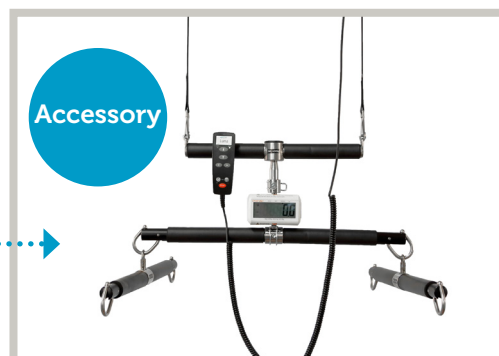



With Luna Twin **room to room** transfer is also possible. Just slide the Luna Twin lift through the unique door connector from Ergolet .



Elite Healthcare Ltd.
Phone: +353 (0) 1 8903932
Fax: +353 (0) 1 8903937

Web: www.elitehealthcare.ie
Email: info@elitehealthcare.ie

Dublin: F1 Swords Enterprise Park,
Feltrim Road, Swords, Co Dublin,
K67 E984, Ireland.

Cork: 602 Harbour Point Business
Park, Little Island, Cork, T45 X624,
Ireland.


Luna Twin

Bariatric solution

elite
healthcare

Technical specification

Max. load of two units	400 kg	Aluminium chassis
Max. load of one unit	275 kg	Electrical and manual emergency lowering
Length of lifting strap	2200 mm	Electrical emergency stop
Weight of one unit	6,7 kg w/o spreader bar	Low battery indicator (visual LED)
Noise without / with max. load	48dB (A) / 52dB (A)	Charging indicators
Height of weigh scale	117 mm	Power on indicators
Full charging time	5 hours	Overload protection
Protection class	IPx4	Emergency brake system

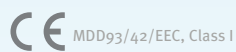

MDD93/42/EEC, Class I

See the technical manual for further specifications. All measurements and numbers are for guidance only and may vary in different situations. Ergolet reserves the right to make changes without any notice. Product photos may be shown with accessories. Talk to your dealer.



Elite Healthcare Ltd.
Phone: +353 (0) 1 8903932
Fax: +353 (0) 1 8903937

Web: www.elitehealthcare.ie
Email: info@elitehealthcare.ie

Dublin: F1 Swords Enterprise Park,
Feltrim Road, Swords, Co Dublin,
K67 E984, Ireland.

Cork: 602 Harbour Point Business
Park, Little Island, Cork, T45 X624,
Ireland.