



## MANGAR Camel Lifting Cushion

The Camel Lifting Cushion will both sit up and lift a fallen person.

### EASY TO FIT

- Lifting cushion is inflated using an Airflo Compressor with a simple hand control
- Simple to use, requiring very little training
- Suitable for indoor and outdoor use
- Can be used independently or assisted

### SIMPLE TO MAINTAIN

- Clean using a proprietary non-abrasive mild liquid cleaner.
- Easy to deflate, fold away and store in the carry bag provided
- Does not need a LOLER certificate
- The material is designed to exceptionally high standards with extraordinary durability, strength & performance in the toughest of environments

### SAFE TO USE

- Lifts the frailest person up to 320 kg from the floor
- Stable and supportive
- Ideal for carers of individuals with neurological disorders who fall regularly

### IMPROVES WELL-BEING

- Provides a safe, dignified lift for a fallen person
- Reduces the risk of musculoskeletal injury for the carer
- Individuals with a cognitive impairment are less distressed when lifted with a Camel
- The backrest gives additional support and is particularly suitable for bariatric patients

### TECHNICAL SPECIFICATIONS

|                                |               |
|--------------------------------|---------------|
| Max user weight                | 320 kg        |
| Max height                     | 56 cm         |
| Min height                     | Flat          |
| Seat depth                     | 47 cm         |
| Total length                   | 112 cm        |
| Width                          | 70 cm         |
| Total product weight           | 6.5 kg        |
| Back/headrest length           | 81 cm         |
| Airflo Compressor              | 24            |
| Lifting performance per charge | up to 4 lifts |



## Features

1. **Easy to setup** - connect the four hoses from the hand control to the corresponding numbers and coloured connectors on the Camel.
2. **The head should lie on the Camel badge** and the body should lie centrally.
3. **The backrest** may be partially inflated before the lower sections.
4. **Made from very strong polyurethane coated nylon material.**
5. **Each layer inflates individually** ideal for single-handed care.

## Accessories

- Slidesheet
- Camel Carry Bag



Product code: MPCA060400



Scan me

**elite**  
healthcare

20 Primeside Park  
Northwest Business Park  
Ballycoolin, Dublin 15. D15 E393

T: 01 8903932  
E: [info@elitehealthcare.ie](mailto:info@elitehealthcare.ie)  
[www.elitehealthcare.ie](http://www.elitehealthcare.ie)

